

# Next<sup>®</sup>

PROPHY PASTE

**CONTAINS:**

NEXT<sup>®</sup> Propphy Paste contains a unique blend of polishing and cleaning agents designed for use by dental professionals in the course of standard dental hygiene procedures. Contains fluoride at a concentration of 1.23% fluoride ion.

**INDICATIONS:**

For cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. NOTE: PROPHYLAXIS PASTES CONTAINING FLUORIDE ARE NOT CONSIDERED A REPLACEMENT FOR OTHER FORMS OF FLUORIDE THERAPY.

**ADMINISTRATION:**

NEXT Propphy Paste is to be used only by individuals professionally trained to perform dental prophylaxis.

**PRECAUTIONS:**

Excessive ingestion of fluoride can cause nausea. NEXT

Prophylaxis Paste should be expectorated and NOT SWALLOWED.

**DIRECTIONS: FOR USE:**

Unit dose cups contain enough NEXT Prophylaxis Paste for a single prophylaxis procedure.

Using a disposable prophyl angle such as PIVOT<sup>®</sup> disposable prophyl angle or a conventional metal angle with a disposable prophyl cup, fill the cup by pressing the face of the cup into the paste while running the angle at low speed. Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

**CAUTION:**

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

**SUGGESTED APPLICATIONS:**

Mint	Cherry	Bubble Gum	Grit	Color	Description
✓	✓	✓	Fine	Gold	Gentle polishing and cleansing ideal for children.
✓	✓	✓	Medium	Blue	Normal polishing, cleansing and plaque removal.
✓	✓	✓	Coarse	Red	Medium to heavy stain and plaque removal.
✓			Extra Course	Green	Heavy stain build-up and plaque removal.

Tropical Fruit	Chocolate Mint	Cinnamon Burst	Grit	Color	Description
✓			Fine	Gold	Gentle polishing and cleansing ideal for children.
✓	✓	✓	Medium	Blue	Normal polishing, cleansing and plaque removal.
✓	✓	✓	Coarse	Red	Medium to heavy stain and plaque removal.
			Extra Course	Green	Heavy stain build-up and plaque removal.

Water-melon	Vanilla	Wintergreen	Grit	Color	Description
✓	✓	✓	Fine	Gold	Gentle polishing and cleansing ideal for children.
✓	✓	✓	Medium	Blue	Normal polishing, cleansing and plaque removal.
✓	✓	✓	Coarse	Red	Medium to heavy stain and plaque removal.
		✓	Extra Course	Green	Heavy stain build-up and plaque removal.

For added convenience, consider the PIVOT Propphy Pack which combines in one package the performance and reliability of a PIVOT disposable prophyl angle with the polishing power of NEXT propphy paste in the most popular flavors and grits. Ask your dental dealer for details.

For assorted flavors in a single box of NEXT Propphy Paste:

Box	Grit	Description
Box 200 Cups	Medium	Kid's Variety Pack
Box 200 Cups	Coarse	Kid's Variety Pack
Box 200 Cups	Medium	Adult Variety Pack
Box 200 Cups	Coarse	Adult Variety Pack

Flavors
Mint, Cherry and Bubble Gum
Mint, Cherry and Bubble Gum
Mint, Chocolate Mint, Cinnamon
Mint, Chocolate Mint, Cinnamon