

CONTAINS:

MAXmin[™] Prophy Paste contains a unique blend of polishing and cleaning agents designed for use by dental professionals in the course of standard dental hygiene procedures. Contains fluoride at a concentration of 1.23% fluoride ion. Contains xylitol.

INDICATIONS:

For cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. NOTE: PROPHYLAXIS PASTES CONTAINING FLUORIDE ARE NOT CONSIDERED A REPLACEMENT FOR OTHER FORMS OF FLUORIDE THERAPY. NOT FOR VETERINARY USE.

ADMINISTRATION:

MAXmin Prophy Paste is to be used only by individuals professionally trained to perform dental prophylaxis.

PRECAUTIONS:

Excessive ingestion of fluoride can cause nausea. MAXmin Prophylaxis Paste should be expectorated and NOT SWALLOWED.

This product contains xylitol and is not recommended for veterinary use.

DIRECTIONS FOR USE:

Unit dose cups contain enough MAXmin Prophylaxis Paste for a single prophylaxis procedure.

Using a disposable prophy angle such as PIVOT® Disposable Prophy Angle or a conventional metal angle with a disposable prophy cup, fill the cup by pressing the face of the cup into the paste while running the angle at low speed. Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

CAUTION:

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

SUGGESTED APPLICATIONS:

Mint	Cherry	Grit	Color	Description
•	•	Medium	Blue	Normal polishing, cleansing and plaque removal.
•	•	Coarse	Red	Medium to heavy stain and plaque removal.

Bubble Gum	Variety Pack	Grit	Color	Description
•	•	Medium	Blue	Normal polishing, cleansing and plaque removal.
•	•	Coarse	Red	Medium to heavy stain and plaque removal.